

Food and Drug Administration  
Device Modification – Envoy Patient Monitor:  
Special 510(k) for VitaLogik

---



DEC 20 2005

Date: November 10, 2005

Topic: **510(k) Safety and Effectiveness Summary as per 21 CFR Section 807.92(c)**

**Special 510(k): Device Modification – VitaLogik**

**Establishment Name, Registration Number and Address:**

Name: Mennen Medical Ltd.  
Registration Number: 9611022  
Operator Number: 9011766  
Address: 4 Hayarden Street, Yavne, 81228, Israel  
Postal Address: PO Box 102,  
Rehovot, 76100, Israel  
Tel: +972-8-9323333  
Fax: +972-8-9328510  
Contact person: Micha Oestereich, Regulatory Affairs

To: Food and Drug Administration  
Center for Devices and Radiological Health  
Document Mail Center (HFZ-401)  
9200 Corporate Boulevard  
Rockville MD, 20850

Attn.: Document Control Clerk  
From: Micha Oestereich, Regulatory Affairs

**Product Name**

Proprietary: VitaLogik  
Common: Physiological Patient Monitor  
Mennen Medical Part Number: 641-000-000

**FDA Classification**

Classification Name: Arrhythmia Detector and Alarm  
Classification Number: 21 CFR 870.1025  
Classification: Class III  
Product Code: 74 DSI

**Performance Standards:** None promulgated

**Voluntary Standards:**

\***IEC 60601-1:** Medical Electrical Equipment Part:1 General Requirements for Safety  
File 4745C 1998:12

\***IEC 60601-1-1 (2001)** Medical Electrical Equipment Part 1-1: General Requirements  
for Safety Collateral Standard: Safety Requirements for Medical Electrical Systems

\***IEC 60601-1-2 (2001):** Medical Electrical Equipment Part 1-2: General  
Requirements for Safety - Collateral Standard: Electromagnetic Compatibility -  
Requirements and Tests.

\***IEC 60602-2-27 (1994):**  
Medical electrical equipment, Part 2,  
Requirements for safety of electrocardiograph monitoring equipment.

\***IEC 60601-2-30 (1995):**  
Medical electrical equipment, Part 2 - requirements for safety of automatic  
cycling indirect blood pressure monitoring equipment

\***IEC 60601-2-34 (1994):**  
Medical electrical equipment, Part 2 - Particular requirements for the safety of  
direct blood pressure monitoring equipment

\* **IEC 60601-2-49 (2001):**  
Particular Requirements for the safety of multifunction patient monitoring  
equipment

\* **IEC 60601-1-8 (2003):**  
General requirements for safety-collateral requirements, test & guidance  
for alarm system in medical electrical equipment & medical electrical systems

**Terminology:**

**Envoy Patient Monitor = the predicate device.** The Envoy was cleared for marketing by the FDA in the following 510(k) submittals:

- K974510 – 14<sup>th</sup> April 1998
- K983864 – 8<sup>th</sup> October 1999
- K000563 – 17<sup>th</sup> May 2000
- K001120 – 8<sup>th</sup> May 2001
- K011784 – 16<sup>th</sup> August 2001
- K022168 – 1<sup>st</sup> August 2002

**VitaLogik = Subject of this Special 510(k).** The VitaLogik is a modified device to the Envoy Patient Monitor

**1. Device Description: VitaLogik Monitor**

The VitaLogik is a configured multi-parameter physiological patient monitor, based on the hardware and software of the Mennen Medical Envoy monitor, without the module rack and modules.

The front end electronic is composing the modules hardware and software into the chassis of the Bed Side Computer =BSC. The input connectors are incorporated in the front panel of the BSC

It is offered in two basic options: Non-Invasive monitor and Full monitor.

In the Non-Invasive version the vital signs are ECG, NIBP, SpO2 and Temperature. The Full monitor includes also two invasive Blood Pressures and Cardiac Output. Both have EtCO2 as an option.

**Functional Description of the VitaLogik**

The VitaLogik is a configured (no modules) monitor, based on the Envoy monitor, hardware and software, without the module rack and modules.

It measures vital signs such as ECG/Heart rate, NIBP, SpO2, Temperature Cardiac output and EtCO2 as an option.

It will be offered in two basic options: Non-Invasive monitor ECG/Heart rate, NIBP, SpO2, Temperature and Full monitor that includes also two invasive BP and Cardiac output. Both will have EtCO2 as an option.

It will have one serial input for vendor devices.

The VitaLogik uses identical display and patient data as does the Envoy. The Ensemble central station and the Enguard remote monitor can both view the VitaLogik as well as the Envoy.

The VitaLogik has serial input for interface with other vendor devices in the same way that does the UIM module of the Envoy.

### Non Invasive VitaLogik version: Vital signs parameters

- Diagnostic 7 or 12 Lead ECG
- Non-invasive Blood Pressure
- Pulse Oximetry (SpO<sub>2</sub>)
- Temperature
- EtCO<sub>2</sub> (optional)

### Full VitaLogik version: Vital signs parameters

- Diagnostic 7 or 12 Lead ECG
- Non-invasive Blood Pressure
- Pulse Oximetry (SpO<sub>2</sub>)
- Temperature
- 2 Invasive Blood Pressure channels
- Cardiac Output
- EtCO<sub>2</sub> (optional)

### VitaLogik Options:

- Non Invasive monitor
- Full monitor
- EtCO<sub>2</sub>

### Main components of the VitaLogik:

The VitaLogik system consists of:

- (A)** a **Bed side computer** and
- (B)** a **Display**

**(A)** The **Bed side computer** acquires, processes, and converts vital signs from the patient into waveforms and digital signals.

The VitaLogik can acquire the following physiological signals of the patient:

- ECG – Waveform and measures Heart Rate, ST and Arrhythmia
- Blood Pressure – Waveform and measures Systole, Diastole and Mean Pressure
- Temperature – As a numeric value in C° or F°
- SpO<sub>2</sub> – Photoplethysmographic waveform and numeric value of the oxygen saturation and pulse rate

- NIBP – Systolic, Diastolic and Mean pressure with measuring time stamp
- EtCO2 – EtCO2, inCO2 and Respiration Rate

(B) The **Display** is used to display the measurement and waveforms, and alarms. It does not have any control function.

## INDICATIONS FOR USE

VitaLogik is intended for use as a multiparameter physiological patient monitoring system.

The VitaLogik can monitor ECG/heart rate, two invasive blood pressure channels, two temperature channels, pulse oximetry, respiration, non-invasive blood pressure and EtCO2. This effectively allows the VitaLogik to monitor a wide-range of adult, pediatric and neonatal patient conditions, in many different areas of the hospital.

Functions include display of multiparameter waveforms, vital signs, alarm & status messages.

The Mennen Medical VitaLogik is intended for sale as a system for monitoring and recording patient information or any in-hospital application requiring patient monitoring.

The following are examples of intended clinical applications:

- Critical Care Patients
- Cardiac Step-down/Telemetry Units
- Emergency Departments
- Intra-operative (Anesthesia) Monitoring
- Post Anesthesia Care

## 2. Substantial Equivalence: VitaLogik versus Envoy

### Comparison: Envoy with VitaLogik

The following tables summarize and compare data on the Envoy (predicate device –K001120) to the subject of this Special 510(k) submittal, the VitaLogik.

	Envoy	VitaLogik
Part/Option Number	550-010-017	641-000-000
<b>Input Circuit Parameters</b>		
<b>Chassis Leakage Current</b>	All patient signal inputs fully isolated (<50 $\mu$ A) Meets or exceeds ANSI standard: "Safe Current Limits for Electromedical Aparatus," (SCLE) Dec, 1978 item 2.1.1.	Same
<b>Degree of protection against electrical shock</b>	Type CF and BF. ECG, IBP and CO = CF NIBP and SpO2 = BF	Same
<b>Electrosurgical Interference Suppression</b>	Yes	Same
<i>ECG</i>	3/5/6/12 Lead	Same
<b>Input Impedance:</b>	Typical 20 M $\Omega$ Minimum greater than: 5 M $\Omega$ differential, DC to 10 Hz; 2.5 M $\Omega$ differential 10 to 100 Hz 3 M $\Omega$ differential at 10 Hz	Same
<b>Input Dynamic Range:</b>	$\pm$ 5mV p-p at a rate up to 320mV/sec, as per ANSI/AAMI EC13 <sup>(8)</sup> Para. 3.2.9.1.	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	<b>Envoy</b>	<b>VitaLogik</b>
<b>Input offset</b>	± 300mV, as per ANSI/AAMI EC13 Para. 3.2.9.1.	Same
<b>Noise:</b>	Less than 30 µ V p-p referenced to input	Same
<b>Defibrillator Protection:</b>	Up to 5 KV. Amplifier Recovery time: < 3 seconds	Same
<b>Gain:</b>	Manual selection of 1/4, 1/2, 1, 2, 4, 8 mV/cm	Same
<b>ECG Analog Output:</b>	1 Volt / mVolt	Same
<b>Sampling Rate and Resolution</b>	Sampling rate: 640Hz Resolution: 22 bit	Same Same
<b>Frequency Response</b>	Diagnostic : 0.05 to 150 Hz Monitoring: 0.5 to 40 Hz Exercise: 1.0 to 25 Hz ST: 0.05 to 40 Hz	Same
<b>QRS Detection Range</b>	Height: 0.25 to 5.0 millivolt Width: 70 to 120 mSec	Same
<b>Heart Rate Counting</b>	Range: 20 to 300 BPM Accuracy: ± 2 BPM Note: Values below 20 are forced to zero.	Same
<b>Heart Rate Alarm Settings</b>	High and low rate: 20 - 250 BPM non-overlapping	Same
<b>Leads analyzed for Heart Rate and Arrhythmia Configuration</b>	Top two displayed	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	Envoy	VitaLogik
<b>ECG Leads:</b>		Same
3 Lead cable	-- I or II or III	
5 Lead cable	-- I, II, III, aVR, aVL, aVF, V	
6 Lead cable	-- I, II, III, aVR, aVL, aVF, Va, Vb	
12 Lead module	-- I, II, III, aVR, aVL, aVF, V1-V6	
<b>Common Mode Rejection</b>	120 dB, minimum	Same
<b>Lead Fault Detection</b>	Based on impedance	Same
<b>Pacemaker Detection and Rejection</b>	Amplitude: 2 mV to 700 mV Width: 0.1 ms to 2.0 ms Pacemaker flag inserted into displayed waveform	Same
<b>Data Storage :</b>	<ul style="list-style-type: none"> <li>• Beat notification</li> <li>• RR Interval</li> <li>• Heart Rate</li> <li>• ST values</li> <li>• Arrhythmia</li> <li>• Alarms</li> <li>• Parameter settings</li> <li>• Cycle time and measurement time markers</li> <li>• All measurements for 36 hrs</li> <li>• S/D/M and pulse rate values</li> <li>• Alarm event markers</li> </ul>	Same
<b>Respiration</b>		
<b>Lead Selection</b>	RA-LA or RA-LL	Same
<b>Respiration Sensitivity Range</b>	0.2 ohm to 5.0 ohm	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	Envoy	VitaLogik
<b>Impedance Range</b>	100 to 3,000 ohm @ 65 kHz	Same
<b>Respiration Rate Counting Range</b>	8 to 150 BPM	Same
<b>Respiration Frequency Response</b>	0.13 to 2.5 Hz (-3 dB)	Same
<b>Sampling rate:</b>	38 Hz	Same
<b>Respiration Alarm Settings</b>	Low rate: 0 - 150 BPM High rate: 8 - 150 BPM Apnea: User configurable Cardiac coincidence alarm	Same
<b>Data Storage :</b>	<ul style="list-style-type: none"> <li>• Respiration rate</li> <li>• Respiration rate Alarms</li> <li>• Apnea alarms</li> <li>• Waveform labels and annotations</li> <li>• Instantaneous resp. rate</li> <li>• Alarm event markers</li> </ul>	Same
<b>BP</b>	LED for function indication	Same
<b>Transducer Excitation Voltage</b>	+5 VDC Separate excitation driver for each channel	Same
<b>Site Labels:</b>	BPX, ART, PAP, CVP, RAP, LAP, ICP	Same
<b>Input Sensitivity</b>	5 microVolt/Volt/mmHg	Same
<b>Dynamic Range</b>	-- Pressure range: -50 to +300 mmHg -- Zero range: ±200 mmHg -- Total dynamic range: -200 to +450 mmHg	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	Envoy	VitaLogik
<b>Zero Accuracy</b>	$\pm 0.2$ mmHg	Same
<b>Zero Drift</b>	Less than $\pm 0.2$ mmHg in 24 hours, at constant temperature)	Same
<b>Blood Pressure Accuracy</b>	$\pm 2$ mmHg or $\pm 2\%$ , whichever is greater, exclusive of transducer	Same
<b>Blood Pressure Linearity</b>	within 1% across entire range	Same
<b>Waveform Frequency Response</b>	0 - 15 Hz	Same
<b>Sampling Rate</b>	320 Hz	Same
<b>Fault Detection</b>	<ul style="list-style-type: none"> <li>• Shorted transducer</li> <li>• Transducer in/out</li> <li>• Cable out</li> </ul>	
<b>Data Storage</b>	Systolic, Diastolic and Mean Alarms	Same
<b>Cardiac Output and Temperature</b>		
<b>Adapter and Compatibility Cables</b>	-- COSet <sup>a</sup> Interface cable -- Ice Bath Cardiac Output interface cable -- Dual temperature interface cable (ysi-400)	Same
<b>Temperature Range</b>	Blood temperature: 27°C to 45°C (80.6° to 113° F) Injectate temperature: 0°C to 25°C (32° to 77° F) Body temperature: 0°C to 43°C (32° to 109.4°F)	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for Vitalogik

	Envoy	VitaLogik
<b>Accuracy</b>	Blood temperature: ±0.1°C (32.18°F)  Injectate temperature: ±0.1°C (32.18°F)	Same
<b>Excitation</b>	10µA, maximum	Same
<b>Frequency Response</b>	0 to 15 Hz	Same
<b>Cardiac Output Determination Range</b>	0 to 20 liters per minute	Same
<b>Injectate Volumes</b>	1, 3, 5, and 10cc	Same
<b>Computation Constants</b>	Table built as new values are used	Same
<b>Displayed Data</b>	<ul style="list-style-type: none"> <li>• Cardiac Output,</li> <li>• Cardiac Index,</li> <li>• Stroke Volume,</li> <li>• Stroke Volume Index,</li> <li>• Blood Temperature,</li> <li>• Injectate Temperature,</li> <li>• Trial Number</li> </ul>	Same
<b>Data Storage : – In Cardiac Output mode</b>	<ul style="list-style-type: none"> <li>• Cardiac Output</li> <li>• Hemodynamic Calculation results</li> <li>• Measuring time</li> </ul>	Same
<b>Data Storage – In Two Temp mode:</b>	<ul style="list-style-type: none"> <li>• Temperatures</li> <li>• Delta-Temp</li> <li>• Temperature Alarms</li> </ul>	Same
<b>Non-Invasive Blood Pressure</b>	LED for function indication  Oscillometric Method	Same
<b>Displayed Parameters</b>	Systolic, Diastolic, Mean pressure values  Time of last measurement, measurement Interval,	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	Envoy	VitaLogik
<b>Inflation Rate</b>	Within 5 sec.	Same
<b>Initial Inflation:</b>	150 mmHg (adult) 120 mmHg (pediatric).	Same
<b>Cycle Times</b>	Deflation time (typical): 30 sec. BP time-out: 60 - 180 sec	Same
<b>Measurement Ranges, Adult (in mmHg)</b>	Systolic: 25 to 255 Diastolic: 10 to 220 Mean: 18 to 235	Same
<b>Measurement Ranges, Neonatal (in mmHg)</b>	Systolic: 20 to 135 Diastolic: 5 to 110 Mean: 10-125	Same
<b>Modes:</b>	Auto, Manual, STAT	Same
<b>Pressure (Transducer) Accuracy</b>	$\pm 3$ mmHg or $\pm 2\%$ , whichever is greater	Same
<b>Heart Rate</b>	40 to 140 BPM	Same
<b>Data Storage :</b>	<ul style="list-style-type: none"> <li>• measurement time markers</li> <li>• S/D/M</li> <li>• Alarm event markers</li> </ul>	
<b>Pulse Oximetry (SpO<sub>2</sub>)</b>	Masimo or Nelcor Technology	Same
<b>Saturation Range</b>	1% to 100% SpO <sub>2</sub>	Same
<b>SpO<sub>2</sub> Accuracy</b>	% SpO <sub>2</sub> $\pm 1$ standard deviation	Same
<b>Pulse Rate Range</b>	20 to 250 BPM $\pm 3$ BPM	Same
<b>Saturation alarm limits:</b>	50% to 100%	Same
<b>Data Storage :</b>	<ul style="list-style-type: none"> <li>• Heart rate and O<sub>2</sub> Sat</li> <li>• Alarms</li> </ul>	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	<b>Envoy</b>	<b>VitaLogik</b>
<b>End Tidal CO2 Microstream</b>	LED for function indication Air outlet	Same
<b>Displayed Data</b>	Waveform labels and annotations EtCO2, inCO2 and respiration rate values	Same
<b>CO2 Display Range:</b>	0-100 mmHg	Same
<b>Typical Accuracy: CO2</b>	±2 mmHg for CO2 range of 0-38 mmHg ±5% for CO2 range of 39-99 mmHg + 0.08% for every 1 mmHg above 38 mmHg	Same
<b>Typical Accuracy: Respiration Rate</b>	0 – 70 bpm +/- 1 bpm 71 – 120 bpm +/- 2 bpm 121 – 150 bpm +/- 3 bpm	Same
<b>Rise Time</b>	190 msec (10% - 90%)	Same
<b>Delay Time</b>	2.7 Sec (10% - 90%) typical	Same
<b>CO2 Alarm Limits:</b>	0 to 100 mmHg/ 0 to 10% 0-15 kPA	Same
<b>Accuracy:</b>	for % measurement: 0.1% for mmHg measurement: 1mm	Same
<b>Respiration Rate Alarm Limits:</b>	Neonatal: 0 to 150 BMP Adult: 0-50 BPM	Same
<b>Sidestream Flow Rate:</b>	50 ml/min. nominal	Same
<b>Start-up Time:</b>	30 sec. typical	Same
<b>Automatic Compensation:</b>	At least once per hour	Same
<b>Ambient Temperature:</b>	0-65° C Sidestream	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	Envoy	VitaLogik
<b>Humidity:</b>	10-95% RH, non-condensing	Same
<b>Barometric Pressure:</b>	430-795 Hg (-1250 to 15,000 ft.)	Same
<b>Data Storage :</b>	<ul style="list-style-type: none"> <li>• EtCO2, inCO2 and Respiration Rate values</li> <li>• Alarms</li> <li>• Apnea Alarm</li> </ul>	Same
<i>Universal Input</i>	Interface to other vendor device protocols in RS232.	Same
<b>Number of inputs</b>	One on Uniport module Three on Multiport	One only
<b>Cable In/Out Detection</b>	Yes	Yes
Provides <b>electrical isolation</b> between the monitor and external device/s	Yes	Yes
<b>Displays clinical and technical alarms</b> from external device/s	Yes	Yes
<i>Clinical Software Features</i>		
<i>Waveform Display</i>	<ul style="list-style-type: none"> <li>• 175 mm horizontal area</li> <li>• 7 sec. @ 25mm/sec</li> <li>• Up to 8 traces</li> <li>• Overlapping traces</li> <li>• Expanded display</li> </ul>	Same
<i>Numeric Display</i>	85 mm horizontal area <ul style="list-style-type: none"> <li>• 25mm Heart Rate</li> <li>• 20mm SpO2</li> <li>• "Big Numbers"</li> </ul>	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	<b>Envoy</b>	<b>VitaLogik</b>
<i>Default Alarms</i>	<ul style="list-style-type: none"> <li>• User defined</li> <li>• Fixed or calculated values</li> </ul>	Same
<i>Data Display</i>	<ul style="list-style-type: none"> <li>• Tabular charts</li> <li>• Graphic trends</li> </ul>	Same
<b><u>Electrical Specifications</u></b>		
<i>Main Processing Unit</i>	AC Power Input 90-132/180-264 VAC, single phase, at 47-63 Hz. Maximum current: Dual fuse: each 3.15A, 250V. Slow Blow	Same
<b>AC Power Output for Local Display</b>	90-132/180-264 VAC at 47-63 Hz, single phase. Comes through the MPU power switch. Maximum Output power: 130W	Same
<i>Display Monitor</i>	15", 17", 20" and flat screen (15", 18") are available. Complies with Part 15 of the FCC Rules	Same
<b><u>Environmental Spec.</u></b>		

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	<b>Envoy</b>	<b>VitaLogik</b>
<b>Operating Conditions</b>	Temperature: +5°C to +40°C Humidity: 10 to 95 percent, non-condensing Altitude: -350 to 3050 meters (-1300 to 10,000 feet) Vibration/Shock: per Mennen Medical: Design for Regulatory	Same
<b>Storage Conditions</b>	Temperature: -15 °C to +60 °C Humidity: 10 to 95 percent, non-condensing Altitude: -350 to 5000 meters (-1300 to 17,000 feet)	Same

	<b>Envoy</b>	<b>VitaLogik</b>
<b>Displayed Waveforms</b>		
ECG	Up to 12 lead	Same
BP	Up to 4, separate or superimposed	2 separate or superimposed
Respiration	1	Same
SpO2	1	Same
EtCO2	1	same
<b>Displayed Numeric Parameters</b>		
Heart Rate	Yes	Same
Respiration Rate	Yes	Same
SpO2	Yes	Same

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

	<b>Envoy</b>	<b>VitaLogik</b>
<b>Displayed Waveforms</b>		
ECG	Up to 12 lead	Same
BP	Up to 4, separate or superimposed	2 separate or superimposed
Respiration	1	Same
SpO2	1	Same
EtCO2	1	same
<b>Displayed Numeric Parameters</b>		
BP – Systolic, Diastolic, Mean	Yes	Same
Temperature	2	Same
EtCO2	Yes (optional module)	Optional
<b>Alarm Indications</b>	Yes	Same
<b>Display Functions</b>	<b>Envoy</b>	<b>VitaLogik</b>
Change ECG Lead Selection	YES	Same
Display of Arrhythmia Information	YES	Same
Data Review: Trends - Graphic	YES	Same
Data Review: Chart – Tabular	YES	Same
User defined Configuration Setup	YES	Same
User defined Default Settings	YES	Same
<b>Accessories</b>	Envoy accessories	Same
GUI	Menu driven	same

**Comparison**

**Hardware comparison**

<b>Item</b>	<b>Envoy</b>	<b>VitaLogik</b>
Front End electronics	Module rack	Integrated front end electronics
Front Panel keys	10	3
Quicknobe	Yes	Same
Main screen	Yes	Same
Escape	Yes	Same
Vital Signs - Orange	Yes	No
Patient data - Orange	Yes	No
Setup - Orange	Yes	No
Timer - Yellow	Yes	No
Print – Green	Yes	No
Record – Green	Yes	No
Event – Green	Yes	Yes
Freeze – Green	Yes	No
Silence – Red	Yes	Yes
Alarm Off – Red	Yes	Yes

Food and Drug Administration  
 Device Modification – Envoy Patient Monitor:  
 Special 510(k) for VitaLogik

### Main Menu items comparison

Menu item	Envoy	VitaLogik
Vital Signs	Dynamic list of vital signs per available modules	Fixed list of vital signs
ECG	Yes	Yes
Respiration	Yes	Yes
NIBP	Yes	Yes
Temperature	Yes	Yes
SpO2	Yes	Yes
BP	Yes	Yes
CO	Yes	Yes
EtCO2	Optional	Optional
Spirometry	Yes	No
EEG	Yes	No
Patient data	Same	Same
Setup	Same	Same
Setup -- Software version	Shows software version and module rack, setting.	Show only software version
Setup – Parallel port	Yes	No
System Setup	Same	Same
System Setup– Parallel port setup	Yes	No

### 3. Similarities and Differences in Design:

#### VitaLogik versus Envoy

The following technological and other characteristics/features apply to both the VitaLogik and the Envoy.

- Intended for use in hospitals
- Do not change the functionality of the monitor
- Isolated inputs for vital signs sensors
- ECG amplifier front end with defibrillator protection
- Invasive BP input circuit
- Non Invasive BP measurement
- SpO2 measurement
- Selectable filters for ECG
- Analog output for ECG and BP
- Display of vital signs and physiological waveforms
- Same GUI and same menus
- Monitoring at central nurse station

The major differences between the VitaLogik and the Envoy are

- The VitaLogik does not have a module rack
- The VitaLogik is a configured monitor with a fixed set of parameters
- The VitaLogik front panel is different from the Envoy
- The VitaLogik has only 5 hardware keys as compared to 12 on the Envoy panel

The VitaLogik monitor provides access to the functions of the Hardware keys, which were omitted, via the menus. On the original Envoy monitor the Hardware keys were used as a shortcut to the same menu items, thus functionality is not reduced. If one or more of the missing hardware keys in the VitaLogik are frequently used in a given setup the missing hardware key can be replaced by software Quickeys available on both the Envoy and it's modified version VitaLogik.

For example on the Envoy there is a hardware key to 'Patient Data'. On the VitaLogik the 'Patient Data' can be reached via the Main menu, or a software Quicknob can be create as a shortcut to 'Patient Data'.

We submit that the change from Module rack monitor to configured monitor with the same menu and display does not amount to a change in the “fundamental scientific technology” of the Envoy and does not disqualify the VitaLogik from being the subject of a Special 510(k).

The example brought down in 21 CFR 862.9 of a change in the “fundamental

scientific technology” (“a surgical instrument cuts tissue with a laser beam rather than with a sharpened metal blade”), involves a radical change in the method and type of treatment given to the patient.

The VitaLogik SW is a brunch of the Envoy SW that takes into account the replacement of the module rack and modules into a configured monitor

The following table compares the major software element and/or changes done in the VitaLogik vs. the Envoy:

SW Component	Envoy	VitaLogik
Display	All waveforms and numeric vital sings	Same
Operating System	QNX4	Same
GUI	Same	Same
Menus	Full set	Same
Vital signs	Depend on available modules	Fixed

**Conclusion of comparison of technological characteristics:**

We consider the VitaLogik monitor to be substantially equivalent to the Envoy monitor and we submit that any differences between the two systems

- fall within the scope of a Special 510(k) Device Modification and
- do not raise any new issues of safety and effectiveness

**Testing**

The VitaLogik has been subject to extensive safety and performance testing. Final testing for the system included various performance tests designed to ensure that the device meets all functional requirements and performance specifications. Safety testing and EMC testing were performed by an independent testing laboratory to ensure that the device complies with applicable industry and safety standards.

**Indications for Use**

Food and Drug Administration  
Device Modification – Envoy Patient Monitor:  
Special 510(k) for VitaLogik

---

The Indications for Use of the VitaLogik remains basically the same at those set out for the Envoy (predicate device), K001120. See page 16 below for the Indications for Use.

## INDICATIONS FOR USE

VitaLogik is intended for use as a multiparameter physiological patient monitoring system.

The VitaLogik can monitor ECG/heart rate, two invasive blood pressure channels, two temperature channels, pulse oximetry, respiration, non-invasive blood pressure and EtCO<sub>2</sub>. This effectively allows the VitaLogik to monitor a wide-range of adult, pediatric and neonatal patient conditions, in many different areas of the hospital.

Functions include display of multiparameter waveforms, vital signs, alarm & status messages.

The Mennen Medical VitaLogik is intended for sale as a system for monitoring and recording patient information or any in-hospital application requiring patient monitoring.

The following are examples of intended clinical applications:

- Critical Care Patients
- Cardiac Step-down/Telemetry Units
- Emergency Departments
- Intra-operative (Anesthesia) Monitoring
- Post Anesthesia Care



DEC 20 2005

Food and Drug Administration  
9200 Corporate Boulevard  
Rockville MD 20850

Mennen Medical, Ltd  
c/o Ms. Micha Oestereich  
QA & Regulatory Affairs Manager  
P.O. Box 102  
Rehovot, 76100  
ISRAEL

Re: K052288

Trade Name: VitaLogik

Regulation Number: 21 CFR 870.1025

Regulation Name: Arrhythmia Detector and Alarm (including ST-segment measurement and alarm)

Regulatory Class: Class III (three)

Product Code: DSI

Dated: November 16, 2005

Received: November 22, 2005

Dear Ms. Oestereich:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050. This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,



Brad D. Zuckerman, M.D.

Director  
Division of Cardiovascular Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known): K052288

Device Name: VitaLogik

### Indications For Use:

VitaLogik is intended for use as a multiparameter physiological patient monitoring system.

The VitaLogik can monitor ECG/heart rate, two invasive blood pressure channels, two temperature channels, pulse oximetry, respiration, non-invasive blood pressure and EtCO<sub>2</sub>. This effectively allows the VitaLogik to monitor a wide-range of adult, pediatric and neonatal patient conditions, in many different areas of the hospital.

Functions include display of multiparameter waveforms, vital signs, alarm & status messages.

The Mennen Medical VitaLogik is intended for sale as a system for monitoring and recording patient information or any in-hospital application requiring patient monitoring.

The following are examples of intended clinical applications:

- Critical Care Patients
- Cardiac Step-down/Telemetry Units
- Emergency Departments
- Intra-operative (Anesthesia) Monitoring
- Post Anesthesia Care

Prescription Use   x    
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use \_\_\_\_\_  
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

---

Concurrence of CDRH, Office of Device Evaluation (ODE)

Bhinnaman  
(Division Sign-Off)  
Division of Cardiovascular Devices  
510(k) Number K052288