

MAY 4 2006

Exhibit A 510(K) SUMMARY

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR §807.92.

The assigned 510(k) number is: K060949

Submitter:

Shenzhen Mindray Bio-medical Electronics Co., LTD
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan,
Shenzhen, 518057, P. R. China

Tel: +86 755 2658 2888

Fax: +86 755 2658 2680

- **Contact Person:**

Li Dongling

Shenzhen Mindray Bio-medical Electronics Co., LTD
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen, 518057, P. R. China

- **Date Prepared:**

Mar 28, 2006

Name of the device:

- **Trade/Proprietary Name:**

DP-6600 Digital Ultrasonic Diagnostic Imaging System with
added transducers

- **Common Name:** Diagnostic Ultrasound System and Transducers

- **Classification**

Regulatory Class: II

Review Category: Tier II

21CFR 892.1560 Ultrasonic Pulsed Echo Imaging System (90-IYO)

21CFR 892.1570 Diagnostic Ultrasound Transducer (90-ITX)

0023

Legally Marketed Predicate Device:

- K052113 DP-6600 Digital Ultrasonic Diagnostic Imaging System
- K992663 SSD-5000 Diagnostic Ultrasound System, Aloka Co. Ltd.
- K042540 IU22 Diagnostic Ultrasound System, Phillips Ultrasound, Inc.

Description:

The DP-6600 Digital Ultrasonic Diagnostic Imaging System with added transducers is a general purpose, portable, software controlled, ultrasound diagnostic system. This ultrasonic device is designed to project ultrasound waves into body tissue and to present the returned echo information on the monitor. The resulting information is displayed in B-Mode, M-Mode, or in the combined mode (i.e. B/M-Mode). This system is a Track 1 device that employs an array of probes that include linear array and convex linear array with a frequency range of approximately 2 MHz to 10 MHz. Changing acoustic transducers of probes and addition of some transducers enriches the diversity of the product's configuration and expands the freedom of choices for customers. At the same time, new software functions have been added.

Statement of intended Use:

The system is a general-purpose, fully digital ultrasound system for abdominal, gynecologic and obstetric, small parts, and cardiac applications.

The system is intended to use for the following type of studies: fetal organ, abdominal, pediatric, small organs, neonatal cephalic, cardiac, transvaginal, peripheral vascular, and musculo-skeletal (both conventional and superficial). This device is intended to adult, pregnant woman, pediatric and neonate. The system is a prescription device intended to be used by or on the order of a physician or similarly qualified health care professional. This Device is not intended for home use.

Technological Characteristics:

The DP-6600 digital ultrasonic diagnostic imaging system with added transducers incorporates the same fundamental technology as the predicate devices. The device has been tested as Track 1 Device per the FDA Guidance document "Information for Manufacturers Seeking Marketing Clearance of

Diagnostic Ultrasound Systems and Transducers” issued September 1997. The acoustic output is measured and calculated per NEMA UD 2 Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment: 2004. All transducers used with the DP-6600 digital ultrasonic diagnostic imaging system are track 1. All patient contact materials are biocompatible.

The technology characteristics of the DP-6600 digital ultrasonic diagnostic imaging system with these modifications do not affect the safety or efficacy of the device.

Testing:

Laboratory testing was conducted to verify that the DP-6600 digital ultrasonic diagnostic imaging system with added transducers met all design specification and was substantially equivalent to the currently marketed Predicate Device as above. The device has been found to conform to applicable medical device safety standards in regards to thermal, mechanical and electrical safety as well as biocompatibility. Acoustic output is measured and calculated according to “Acoustic Output Measuring Standard for Diagnostic Ultrasound Equipment”

Applicable Standards

The DP-6600 digital ultrasonic diagnostic imaging system with added transducers conforms to the following Standards:

NEMA UD 2 Acoustic Output Measurement Standard for Diagnostic ultrasound Equipment

IEC 60601-1

IEC 60601-1-2

Clinical Test:

No clinical testing was required

Conclusion:

The conclusions drawn from testing of the DP-6600 Digital Ultrasonic Diagnostic Imaging System with added transducers demonstrates that the device is as safe, as effective as well as the legally marketed predicate devices.



MAY 4 2006

Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

Shenzhen Mindray Bio-Medical Electronics Co., Ltd
% Ms. Susan D. Goldstein-Falk
Official Correspondent
mdi Consultants, Inc.
55 Northern Blvd., Suite 200
GREAT NECK NY 11021

Re: K060949

Trade Name: DP-6600 Digital Ultrasonic Imaging System with added Transducers
Regulation Number: 21 CFR 892.1560
Regulation Name: Ultrasonic pulsed echo imaging system
Regulation Number: 21 CFR 892.1570
Regulation Name: Diagnostic ultrasonic transducer
Regulatory Class: II
Product Code: IYO and ITX
Dated: March 20, 2006
Received: April 6, 2006

Dear Ms. Goldstein-Falk:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and we have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

This determination of substantial equivalence applies to the following transducers intended for use with the DP-6600 Digital Ultrasonic Imaging System with added Transducers, as described in your premarket notification:

Transducer Model Number

65C15EA
75L60EA



Protecting and Promoting Public Health

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This determination of substantial equivalence is granted on the condition that prior to shipping the first device, you submit a postclearance special report. This report should contain complete information, including acoustic output measurements based on production line devices, requested in Appendix G, (enclosed) of the Center's September 30, 1997 "Information for Manufacturers Seeking Marketing Clearance of Diagnostic Ultrasound Systems and Transducers." If the special report is incomplete or contains unacceptable values (e.g., acoustic output greater than approved levels), then the 510(k) clearance may not apply to the production units which as a result may be considered adulterated or misbranded.

The special report should reference the manufacturer's 510(k) number. It should be clearly and prominently marked "ADD-TO-FILE" and should be submitted in duplicate to:

Food and Drug Administration
Center for Devices and Radiological Health
Document Mail Center (HFZ-401)
9200 Corporate Boulevard
Rockville, Maryland 20850

This letter will allow you to begin marketing your device as described in your premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus permits your device to proceed to market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>

If you have any questions regarding the content of this letter, please contact Andrew Kang, M.D. at (301) 594-1212.

Sincerely yours,


for Nancy C. Brogdon
Director, Division of Reproductive,
Abdominal and Radiological Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

Diagnostic Ultrasound Indications for Use Form

System x Transducer

Model: DP-6600

510(k) Number(s) K060949

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application	Mode of Operation									
	A	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Color Velocity Imaging	Combined (specify)	Other (specify)
Ophthalmic										
Fetal		P	P						P	
Abdominal		P	P						P	
Intraoperative (specify)										
Intraoperative Neurological										
Pediatric		P	P						P	
Small organ(specify)		P	P						P	
Neonatal Cephalic		P	P						P	
Adult Cephalic										
Cardiac		P	P						P	
Transesophageal										
Transrectal		P	P						P	
Transvaginal		P	P						P	
Transurethral										
Intravascular										
Peripheral Vascular		P	P						P	
Laparoscopic										
Musculo-skeletal Conventional		P	P						P	
Musculo-skeletal Superficial		P	P						P	
Other (specify)										

N=new indication; P=previously cleared by FDA; E=added under Appendix E

Additional comments: Combined mode: B+M

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation(ODE)

Prescription USE (Per 21 CFR 801.109)

David A. Segura

 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number K060949

0029

Diagnostic Ultrasound Indications for Use Form

System _____ Transducer _____ *

Model: 65C15EA

510(k) Number(s) K060949

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application	Mode of Operation									
	A	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Color Velocity Imaging	Combined (specify)	Other (specify)
Ophthalmic										
Fetal										
Abdominal										
Intraoperative (specify)										
Intraoperative Neurological										
Pediatric		N	N						N	
Small organ(specify)										
Neonatal Cephalic		N	N						N	
Adult Cephalic										
Cardiac										
Transesophageal										
Transrectal										
Transvaginal										
Transurethral										
Intravascular										
Peripheral Vascular										
Laparoscopic										
Musculo-skeletal Conventional										
Musculo-skeletal Superficial										
Other (specify)										

N=new indication; P=previously cleared by FDA; E=added under Appendix E

Additional comments: Combined mode: B+M

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation(ODE)

Prescription USE (Per 21 CFR 801.109)



(Division Sign-Off)
Division of Reproductive, Abdominal,
and Radiological Devices
 510(k) number K060949

0030

Diagnostic Ultrasound Indications for Use Form

System _____ Transducer _____ x _____

Model: 75L60EA

510(k) Number(s) K060949

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application	Mode of Operation									
	A	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Color Velocity Imaging	Combined (specify)	Other (specify)
Ophthalmic										
Fetal										
Abdominal										
Intraoperative (specify)										
Intraoperative Neurological										
Pediatric										
Small organ(specify)		N	N						N	
Neonatal Cephalic		N	N						N	
Adult Cephalic										
Cardiac										
Transesophageal										
Transrectal										
Transvaginal										
Transurethral										
Intravascular										
Peripheral Vascular		N	N						N	
Laparoscopic										
Musculo-skeletal Conventional		N	N						N	
Musculo-skeletal Superficial		N	N						N	
Other (specify)										

N=new indication; P=previously cleared by FDA; E=added under Appendix E

Additional comments: Combined mode: B+M

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation(ODE)

Prescription USE (Per 21 CFR 801.109)

David A. Syron
(Division Sign-Off)

Division of Reproductive, Abdominal,
and Radiological Devices

510(k) Number K060949

0031