

5. 510(k) SummaryGeneral Information

Date Compiled	April 5, 2007
Classification	Class II
Trade Name	Acumen Insight Endocardial Visualization System
Submitter	Acumen Medical, Inc. 275 Santa Ana Court Sunnyvale, CA 94085
	Tel: 408-530-1810 Fax: 408-530-1811
Contact	Marybeth Gamber Director, Regulatory Affairs

Intended Use

The Insight Endocardial Visualization System is intended to aid in the visualization of the coronary sinus, foramen ovale and other venous and cardiac anatomy, provide temporary occlusion during a venogram, and to provide a pathway for delivery of transvenous devices to the coronary sinus and coronary vasculature of the heart. The Insight is intended for use in the right heart only.

Predicate Devices

Acumen Insight Endocardial Visualization System	K042381, K062084
Manufactured by Acumen Medical, Inc.	
AcuNav Diagnostic Ultrasound Catheter	K992631
Manufactured by Acuson Corporation	

Device Description

The Acumen Insight Endocardial Visualization System (Insight) is a single-use percutaneous catheter intended to aid in the visualization of the coronary sinus, foramen ovale and other venous side cardiac anatomy, provide temporary occlusion during a venogram, and to provide a pathway for delivery of transvenous devices to the coronary sinus and coronary vasculature of the heart.

Materials

All materials used in the manufacture of the Acumen Insight Endocardial Visualization System are suitable for this use, are identical to the those used in the predicate devices, and have been used in numerous previously cleared products

Testing

As there is no change in the design of the device, additional device testing was not required.

Summary of Substantial Equivalence

Acumen Medical, Inc. believes the Acumen Insight Endocardial Visualization System is substantially equivalent to the predicate product. The intended use, method of operation, methods of construction and materials used, are either identical or substantially equivalent to existing legally marketed predicate product.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

APR - 6 2007

Acumen Medical, Inc.
c/o Marybeth Gamber
Director, Regulatory Affairs
275 Santa Ana Court
Sunnyvale, CA 94085

Re: K070051

Trade/Device Name: Acumen Insight Endocardial Visualization System
Regulation Number: 21 CFR 870.1250
Regulation Name: Catheter, Percutaneous
Regulatory Class: Class II
Product Code: DQY
Dated: March 7, 2007
Received: March 8, 2007

Dear Ms. Gamber:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Page 2 – Ms. Marybeth Gamber

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at 240-276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,



Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

4. Indications for Use Statement

510(k) Number (if known): K070051

Device Name: Acumen Insight Endocardial Visualization System

Indications for Use: The Insight Endocardial Visualization System is intended to aid in the visualization of the coronary sinus, foramen ovale and other venous and cardiac anatomy, provide temporary occlusion during a venogram, and to provide a pathway for delivery of transvenous devices to the coronary sinus and coronary vasculature of the heart. The Insight is intended for use in the right heart only.

Prescription Use X
(Per 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use
(Per 21 CFR 801 Subpart C)

PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)



(Signature Sign-Off)
Division of Cardiovascular Devices
510(k) Number K070051