

Section 5 – 510(k) Summary

Submitter: Possis Medical, Inc.
9055 Evergreen Boulevard NW
Minneapolis, MN 55433-8003 USA

Contact Person: Mike Burnside
Sr. Regulatory Affairs Associate
Phone: (763) 780-4555
Fax: (763) 780-2227
Email: mike.burnside@possis.com

NOV 14 2007

Date Prepared: September 27, 2007

Trade Name: AngioJet[®] Ultra XMI[®], XVG[®], and Spiroflex[®] Thrombectomy Set

Classification: 870.5150

Product Code: DXE

Predicate Device(s): The subject devices are equivalent to the following devices:

- K960970 AngioJet Rapid Thrombectomy System – Pump Set
- K972610 AngioJet LF140 Catheter
- K042874 AngioJet XMI[®] Rheolytic Thrombectomy Catheter
- K071342 AngioJet XPEEDIOR[®] Thrombectomy Set (reference device-design only)
- K072269 (under FDA review) AngioJet DVX[®] Thrombectomy Set (reference device-design only)

Device Description: Each AngioJet Ultra Thrombectomy Set is a sterile, single use, disposable set that includes the Thrombectomy Catheter and Pump in one combined unit. The AngioJet Ultra Thrombectomy Set is used with the AngioJet Ultra Console.

Intended Use:

- The AngioJet Ultra XMI Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries \geq 2mm in diameter.
- The AngioJet Ultra XVG Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries \geq 3mm in diameter.
- The AngioJet Ultra Spiroflex Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries \geq 2mm in diameter.

Functional and Safety Testing: Representative samples of the devices underwent bench testing, including but not limited to mechanical testing, biocompatibility, sterility, comparative testing to demonstrate appropriate functional and performance characteristics.

Conclusion: Possis Medical, Inc. considers the AngioJet Ultra XMI, XVG, and Spiroflex Thrombectomy Sets to be substantially equivalent to the predicate devices listed above. This conclusion is based upon the devices' similarities in functional design, materials, indications for use, and principles of operation.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

NOV 14 2007

Possis Medical, Inc.
c/o Mr. Mike Burnside
Sr. Regulatory Affairs Associate
9055 Evergreen Boulevard NW
Minneapolis, MN 55433-8003

Re: K072769
AngioJet Ultra XMI, XVG, and SpiroFlex Thrombectomy Sets
Regulation Number: 21 CFR 870.5150
Regulation Name: Embolectomy Catheter
Regulatory Class: Class II (Two)
Product Code: DXE
Dated: September 27, 2007
Received: September 28, 2007

Dear Mr. Burnside:


We have reviewed your Section 510(k) premarket notification of intent to market the devices referenced above and have determined the devices are substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the devices, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050. This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Center for Devices and Radiological Health's (CDRH's) Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding postmarket surveillance, please contact CDRH's Office of Surveillance and Biometric's (OSB's) Division of Postmarket Surveillance at 240-276-3474. For questions regarding the reporting of device adverse events (Medical Device Reporting (MDR)), please contact the Division of Surveillance Systems at 240-276-3464. You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

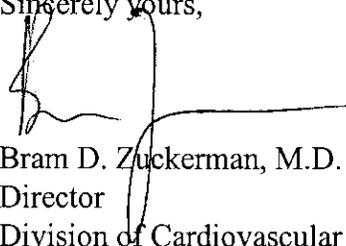

Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K072769

Device Name: AngioJet® Ultra XMI® Thrombectomy Set

Indications for Use:


The AngioJet Ultra XMI Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries ≥ 2 mm in diameter.

Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

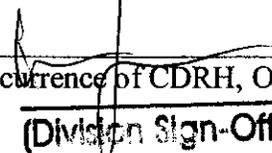
Concurrence of CDRH, Office of Device Evaluation (ODE)

(Division Sign-Off)

Division of Cardiovascular Devices

510(k) Number K072769

Indications for Use

510(k) Number (if known): K072769

Device Name: AngioJet[®] Ultra XVG[®] Thrombectomy Set

Indications for Use:

The AngioJet Ultra XVG Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries ≥ 3 mm in diameter.

Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Indications for Use

510(k) Number (if known): K072769

Device Name: AngioJet[®] Ultra Spiroflex[®] Thrombectomy Set

Indications for Use:

The AngioJet Ultra Spiroflex Thrombectomy Set is intended for use with the AngioJet Ultra Console to break apart and remove thrombus from infra-inguinal peripheral arteries ≥ 2 mm in diameter.

Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)