

510(k) SUMMARY: PATRIOT™ SPACERS

Company: Globus Medical Inc.
2560 General Armistead Ave.
Audubon, PA 19403
(610) 415-9000

Contact: Kelly J. Baker, Ph.D.
Director, Clinical Affairs & Regulatory

Device Name: PATRIOT™ Spacers: JAN 18 2008
Constitution™ PLIF Spacer
Signature™ TLIF Spacer
Continental™ ALIF Spacer
TransContinental™ LLIF Spacer

Classification: Product Code MAX. Class II.
21 CFR §888.3080 Intervertebral body fusion device

Predicate(s): P95002 BAK Interbody Fusion System

Device Description:

The PATRIOT™ Spacers (Constitution™ PLIF, Signature™ TLIF, Continental™ ALIF, and TransContinental™ LLIF Spacers) are lumbar interbody fusion devices used to provide structural stability in skeletally mature individuals following discectomy. Each of the PATRIOT™ Spacers provides a different shape to accommodate various surgical approaches to the lumbar spine (posterior, transforaminal, anterior). The devices are available in various heights and geometric options to fit the anatomical needs of a wide variety of patients. These spacers are to be filled with autogenous bone graft material. Protrusions on the superior and inferior surfaces of each device grip the endplates of the adjacent vertebrae to resist expulsion.

PATRIOT™ Spacers are made from radiolucent polymer, with titanium alloy or tantalum markers, as specified in F2026, F136, F1295, and F560.

Intended Use:

PATRIOT™ Spacers (Constitution™ PLIF, Continental™ ALIF, TransContinental™ LLIF and Signature™ TLIF Spacers) are interbody fusion devices intended for use in patients with degenerative disc disease (DDD) at one or two contiguous levels of the lumbosacral spine (L2-S1). DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These patients should be skeletally mature and have had at least six (6) months of non-operative treatment. In addition, these patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s).

PATRIOT™ Spacers are to be filled with autogenous bone graft material. These devices are intended to be used with supplemental fixation, such as the PROTEX® or REVERE® Stabilization System.

Basis for Substantial Equivalence:

PATRIOT™ Spacers have been evaluated in accordance with the “Class II Special Controls Guidance Document: Intervertebral Fusion Device”, June 12, 2007 and have been found to meet the criteria set forth in the guidance document in terms of indications, design, and performance.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

JAN 18 2008

Globas Medical Inc.
% Kelly J. Baker, Ph.D.
Director, Clinical Affairs & Regulatory
2560 General Armistead Avenue
Audubon, PA 19403

Re: K072970
Trade/Device Name: PATRIOT™ Spacers
Regulation Number: 21 CFR 888.3080
Regulation Name: Intervertebral body fusion device
Regulatory Class: II
Product Code: MAX
Dated: October 19, 2007
Received: October 22, 2007

Dear Dr. Baker:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Page 2 – Kelly J. Baker, Ph.D.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Center for Devices and Radiological Health's (CDRH's) Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding postmarket surveillance, please contact CDRH's Office of Surveillance and Biometric's (OSB's) Division of Postmarket Surveillance at (240) 276-3474. For questions regarding the reporting of device adverse events (Medical Device Reporting (MDR)), please contact the Division of Surveillance Systems at (240) 276-3464. You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,



Mark N. Melkerson
Director
Division of General, Restorative
and Neurological Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Indications for Use Statement

510(k) Number: K072970

Device Name: PATRIOT™ Spacers

Indications:


PATRIOT™ Spacers (Constitution™ PLIF, Continental™ ALIF, TransContinental™ LLIF and Signature™ TLIF Spacers) are interbody fusion devices intended for use in patients with degenerative disc disease (DDD) at one or two contiguous levels of the lumbosacral spine (L2-S1). DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These patients should be skeletally mature and have had at least six (6) months of non-operative treatment. In addition, these patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s).

PATRIOT™ Spacers are to be filled with autogenous bone graft material. These devices are intended to be used with supplemental fixation, such as the PROTEX® or REVERE® Stabilization System.

Prescription Use X OR Over-The-Counter Use
(Per 21 CFR §801.109)

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

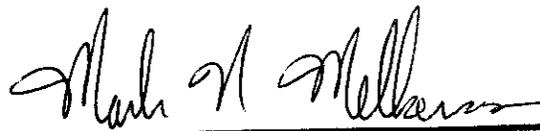

(Division Sign-Off)
Division of General, Restorative,
and Neurological Devices

510(k) Number K072970