STERIS[°]

510(k) Summary For

MAY 2 8 2008

Verify[®] Self-Contained Biological Indicator for Vaporized VH2O2 Sterilization Processes

STERIS Corporation 5960 Heisley Road Mentor, OH 44060 Phone: (216) 354-2600 Fax No: (216) 639-4459

Contact:

John R. Scoville. Fellow Regulatory Affairs Telephone: (440) 392-7330 Fax No: (440) 357-9198

Submission Date: May 16, 2008

STERIS Corporation = 5960 Heisley Road = Mentor, OH 44060-1834 USA = 440-354-2600

1. Device Name

Trade Name:	Verify [®] Self-Contained Biological Indicator for Vaporized VH2O2 Sterilization Processes
Common/usual Name:	Biological Indicator (BI)
Classification Name:	Indicator, Biological Sterilization Process (21 CFR 880.2800, FRC).

2. <u>Predicate Device</u>

STERRAD[®] Cyclesure[™] Biological Indicator, K994055, Feb 13, 2002 STERRAD[®] Cyclesure Biological Indicator, K031226, May 2, 2003 STERRAD[®] Cyclesure Biological Indicator, K071014 May 24, 2007

3. <u>Description of Device</u>

The Verify Self-Contained Biological Indicator (SCBI) for Vaporized VH2O2Sterilization Processes is used by healthcare providers to monitor the Amsco[®] V-PROTM 1 Low Temperature Sterilizer. It is designed to accompany medical devices placed in the sterilizer.

The user places the packaged Verify SCBI for $\overline{VH2O2}$ into the V-PRO 1 Low Temperature Sterilization System and performs a sterilization cycle. After cycle completion, the SCBI can either be immediately activated or it can be held at room temperature for a maximum of 72 hours (3 days) prior to activation.

The SCBI is activated by sealing the vial and rupturing the medium ampoule using the STERIS Verify SCBI HP activator. The activator automatically seals the SCBI vial and releases the growth media.

The activated SCBI is incubated at 55 - 60 °C. The SCBI indicates a pass if the media remains purple and non-turbid. The SCBI indicates a failure of sterilization if the media changes from purple to yellow and/or if the media is turbid.

4. Intended Use

The Verify Self-Contained Biological Indicator for Vaporized VH2O2 Sterilization Processes is intended as a standard method for frequent monitoring of the Amsco V-PRO 1 Low Temperature Sterilization System.

5. Description of Safety and Substantial Equivalence

The Verify SCBI for $\overline{VH2O2}$ has the same or similar intended use, accessories, viable population, resistance characteristics, culture conditions, primary and secondary packaging, and storage conditions as compared to its predicate device the STERRAD CycleSure Biological Indicator.

Summary of Nonclinical Tests:

Test	Result
Viable Population Assay	Pass
	Within Specification
Resistance	Pass
	Within Specification
Growth Inhibition by Carrier and Pack Materials	Pass
	No Inhibition
Holding Time Assessment	Pass
	72 Hour Hold Time Established
Effect of Media Additives	Pass
	No Effect of Additives
Incubation Time Validation	Pass
	4 day Incubation
Effect of Sterilization Process on Recovery Media	Pass
	No Affect
Stability of Biological Read	Pass
	Stable for 7 Days
Controls	Pass
	Viability and Media Sterility Demonstrated
Media Volume	Pass
	Adequate Volume for Incubation Conditions
Throughput Indicator Color Change	Pass
	Color Change Demonstrated under Worst
	Case Exposure Conditions Pass
Stability Evaluation	1 400
	On Going Stability Evaluation Pass
Worst Case Location In V-PRO Sterilization Chamber	Middle of Top Shelf is the Worst Case
	Location



Food and Drug Administration 9200 Corporate Boulevard Rockville MD 20850

Mr. Jack Scoville Fellow, Regulatory Affairs STERIS, Corporation 5960 Heisley Road Mentor, Ohio 44060-1834

MAY 2 8 2008

Re: K073244

Trade/Device Name: Verify Self-Contained Biological Indicator for Vaporized VH202 Sterilization Processes
Regulation Number: 21 CFR 880.2800
Regulation Name: Sterilization Process Indicator
Regulatory Class: II
Product Code: FRC
Dated: April 30, 2008
Received: May 1, 2008

Dear Mr. Scoville:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Page 2 – Mr. Scoville

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0115. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <u>http://www.fda.gov/cdrh/industry/support/index.html</u>.

Sincerely yours,

Chiu Lin. Ph.D.

Director Division of Anesthesiology, General Hospital, Infection Control and Dental Devices Office of Device Evaluation Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K073244

Device Name:

Verify Self-Contained Biological Indicator for Vaporized VH2O2 Sterilization Processes

Indications For Use:

The Verify Self-Contained Biological Indicator for Vaporized $\overline{VH2O2}$ Sterilization Processes is intended as a standard method for frequent monitoring of the Amsco[®] V-PROTM 1 Low Temperature Sterilization System.

Prescription Use _____AND/OROver-The-Counter Use _X_(Part 21 CFR 801 Subpart D)(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

(Division Sign-Off) (Division of Anesthesiology, General Hospital Infection Control, Dental Devices

073244 510(k) Number:

Page 1 of 1

CONFIDENTIAL

May 16, 2008

C-2