

1082847

OCT 29 2008

SECTION II. 510(K) SUMMARY

A. Device Name 17

B. Predicate Device 17

C. Intended Use 17

D. Description 17

E. Principle of Operation / Technology 17

F. Design / Materials 19

G. Specifications 19

H. Performance 19

I. Additional Safety Information 20

J. Substantial Equivalence 20

K. Submitter Information 21

SECTION II. 510(K) SUMMARY

A. DEVICE NAME

Proprietary Name: Pinnacle® ROII Introducer Sheath
Classification Name: Introducer, Catheter
Common Name: Introducer Sheath

B. PREDICATE DEVICE

The predicate device is the Pinnacle® ROII, which is manufactured by Terumo Medical Corporation. The Pinnacle® ROII is cleared through the premarket notification process (K003424).

C. INTENDED USE

The Pinnacle® ROII Introducer Sheath is used to facilitate placing a catheter through the skin into a vein or artery. The Mini Guide Wire is an accessory device which is used for placement of the sheath into the vein or artery. The RADIFOCUS Obturator is also an accessory device which is used by placing it into the sheath to create an occlusion and further provide support to the wall of the indwelling sheath while it remains in place within the vein or artery after removal of a catheter.

Note: This is the same intended use as the Pinnacle® ROII, K003424.

D. DESCRIPTION

The PINNACLE R/O II Introducer Sheath is comprised of an introducer sheath and a dilator. The introducer sheath has a radiopaque marker that is highly visible under fluoroscopy. The marker is a band that is located approximately 5 mm from the sheath's distal tip.

The introducer sheath consists of 2 pieces of ETFE tubing – a long segment and a short tip segment. The band is sandwiched between the two segments and fused together. The band is completely encapsulated within the tubing wall.

The PINNACLE R/O II Introducer Sheath is used to facilitate placement of a catheter through the skin into a vein or artery. A Mini Guide Wire (with Inserter) may be included with the device. The Inserter does not contact blood and is used strictly for guiding the Guide Wire into a cannula or Introducer.

The Mini Guide Wire is inserted through a cannula placed in the patient's blood vessel. The PINNACLE R/O II Introducer Sheath is then inserted over the Mini Guide Wire and into the blood vessel. The Mini Guide Wire is then withdrawn from the vessel. The Dilator maintains the integrity of the Sheath and dilates the blood vessel while the Introducer Sheath is being placed into the vessel. The Dilator can be removed and an appropriate catheter can then be inserted.

The RADIFOCUS Obturator is an accessory device which creates an occlusion when inserted into the Sheath. The Obturator also provides support to the indwelling Sheath after the catheter is removed.

The Sheath, Dilator and Obturator contain bismuth, making these devices visible under fluoroscopy.

E. PRINCIPLE OF OPERATION / TECHNOLOGY

The Pinnacle® ROII Introducer Sheaths operated manually or by a manual process.

F. DESIGN / MATERIALS

The Pinnacle® ROII Introducer Sheath uses the same materials and design as the predicate device. There are no new issues of safety and effectiveness related to design or materials.

G. SPECIFICATIONS

Sheath Sizes: 4Fr. (0.055" nominal lumen size)
5Fr. (0.068" nominal lumen size)
9Fr. (0.122" nominal lumen size)

Sheath Length: 4-110 cm

Dilator Length: 5-110 cm

Guide Wire OD: 0.021" – 0.038"

H. PERFORMANCE

The performance of the Pinnacle® ROII Introducer Sheath is substantially equivalent to the performance of the predicate device K003424. The equivalence was shown through bench testing.

I. ADDITIONAL SAFETY INFORMATION

Sterilization conditions have been validated in accordance with ANSI / AAMI / ISO 11135-1994 to provide a Sterility Assurance Level of 10^{-6} .

Blood contacting materials were tested in accordance with the test recommendations in the FDA General Program Memorandum #G95-1 (5/1/95): Use of International Standard ISO 10993, "Biological Evaluation of Medical Devices – Part I: Evaluation and Testing." The Pinnacle® ROII Introducer Sheath is categorized as "Externally Communicating Device, Circulating Blood, Prolonged Contact (24hrs to 30 days)". The blood contacting materials were found to be biocompatible.

Expiration dating for the Pinnacle® ROII Introducer Sheath will be 30 months.

J. SUBSTANTIAL EQUIVALENCE

The Pinnacle® ROII Introducer Sheath submitted in this 510(k) is substantially equivalent in intended use, design, principle of operation / technology, materials and performance to the Pinnacle® ROII (K003424), which is also manufactured by Terumo Medical Corporation. Differences between the devices do not raise any issues of safety or effectiveness.

K. SUBMITTER INFORMATION

Name and Address

Terumo Medical Corporation
950 Elkton Blvd.
Elkton, MD 21921

Contact Person

Mr. Mark Unterreiner
Sr. Regulatory Affairs Specialist
Ph: 410-392-7213
Fax: 410-398-6079
Email: mark.unterreiner@terumomedical.com

Date Prepared

September 25, 2008



DEPARTMENT OF HEALTH & HUMAN SERVICES

Public Health Service

Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

OCT 29 2008

Terumo Medical Corporation
c/o Mr. Mark Unterreiner
Sr. Regulatory Affairs Specialist
950 Elkton Blvd.
Elkton, MD 21921

Re: K082847
Pinnacle ROII Introducer Sheath
Regulation Number: 21 CFR 870.1340
Regulation Name: Catheter introducer
Regulatory Class: Class II
Product Code: DYB
Dated: September 25, 2008
Received: September 29, 2008

Dear Mr. Unterreiner:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.


Page 2 – Mr. Mark Unterreiner


Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Center for Devices and Radiological Health's (CDRH's) Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding postmarket surveillance, please contact CDRH's Office of Surveillance and Biometrics' (OSB's) Division of Postmarket Surveillance at 240-276-3474. For questions regarding the reporting of device adverse events (Medical Device Reporting (MDR)), please contact the Division of Surveillance Systems at 240-276-3464. You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

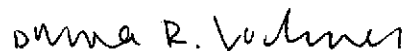

 Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K082847

Device Name: Pinnacle® ROII Introducer Sheath

Indications For Use:

The Pinnacle® ROII Introducer Sheath is used to facilitate placing a catheter through the skin into a vein or artery. The Mini Guide Wire is an accessory device which is used for placement of the sheath into the vein or artery. The RADIFOCUS Obturator is also an accessory device which is used by placing it into the sheath to create an occlusion and further provide support to the wall of the indwelling sheath while it remains in place within the vein or artery after removal of a catheter.

Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 807 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Diana P. Vachner
(Division Sign-Off)
Division of Cardiovascular Devices

510(k) Number K082847