

510(k) SUMMARY OF SAFETY AND EFFECTIVENESS

K083160

1. Applicant

Quality Electrodynamics (QED)
777 Beta Drive
Mayfield Village, OH 44143
Phone (440) 484-2228

NOV - 7 2008

2. Contact

Christie Zydyk, MBA
VP & GM, Regulatory Affairs, Quality Assurance, & Corporate Communications

3. Date prepared:

Revised - October 24, 2008

4. Tradename

1.5T Atlas SPEEDER Head-Neck Coil

5. Common name

Coil, magnetic resonance, specialty

6. Classification

21 CFR 892.1000

7. Equivalent Device

1.5T QD Head Speeder Coil manufactured by USA Instruments Inc. (K041487).

8. Device Description

The Atlas SPEEDER Head-Neck coil is a phased array, multi-channel, receive-only coil. The coil consists of a rigid posterior head base and a flexible posterior chest assembly that are used in combination with various removable rigid assemblies selected based on the anatomy of interest. Foam pads also aid in positioning and enhance patient comfort.

9. Intended Use

For use with a Toshiba Excelart Vantage Atlas or Vantage Titan MR scanner to produce diagnostic images of the brain, cervical spine, and soft tissue and vasculature of the head, neck and upper chest that can be interpreted by a trained physician.

510(k) SUMMARY OF SAFETY AND EFFECTIVENESS

10. Comparison with Predicate Devices

510(k) #	Device	Manufacturer
K041487	1.5T QD Head Speeder Coil	USA Instruments Inc.

The 1.5T Atlas SPEEDER Head-Neck coil and the predicate device are designed for use in conjunction with magnetic resonance scanners to produce diagnostic images of the brain, cervical spine, and soft tissue and vasculature of the head, neck and upper chest that can be interpreted by a trained physician. The 1.5T Atlas SPEEDER Head-Neck coil and the predicate device have similar designs and are constructed of similar materials. The main differences are the number of channels and the number of removable assemblies to accommodate different imaging options.

11. Conclusion

It is the opinion of Quality Electrodynamics that the 1.5T Atlas SPEEDER Head-Neck coil is substantially equivalent to the above-listed legally marketed predicate device. Use of the Quality Electrodynamics coil does not result in any new potential hazards



DEPARTMENT OF HEALTH & HUMAN SERVICES

Public Health Service

Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

Quality ElectroDynamics
% Mr. Mark Job
Responsible Third Party Official
Regulatory Technology Services LLC
1394 25th Street NW
BUFFALO MN 55313

NOV - 7 2008

Re: K083160
Trade/Device Name: 1.5T Atlas SPEEDER Head-Neck Coil
Regulation Number: 21 CFR 892.1000
Regulation Name: Magnetic resonance diagnostic device
Regulatory Class: II
Product Code: MOS
Dated: October 24, 2008
Received: October 27, 2008

Dear Mr. Job:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.


This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at one of the following numbers, based on the regulation number at the top of this letter:

21 CFR 876.xxx	(Gastroenterology/Renal/Urology)	240-276-0115
21 CFR 884.xxx	(Obstetrics/Gynecology)	240-276-0115
21 CFR 894.xxx	(Radiology)	240-276-0120
Other		240-276-0100

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding postmarket surveillance, please contact CDRH's Office of Surveillance and Biometrics' (OSB's) Division of Postmarket Surveillance at 240-276-3474. For questions regarding the reporting of device adverse events (Medical Device Reporting (MDR)), please contact the Division of Surveillance Systems at 240-276-3464. You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

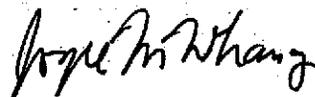

Joyce M. Whang, Ph.D.
Acting Director, Division of Reproductive,
Abdominal, and Radiological Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K083160

Device Name:

1.5T Atlas SPEEDER Head-Neck Coil

Indications for Use:

For use with a Toshiba Excelart Vantage Atlas or Vantage Titan MR scanner to produce diagnostic images of the brain, cervical spine, and soft tissue and vasculature of the head, neck and upper chest that can be interpreted by a trained physician.


Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE OF
NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

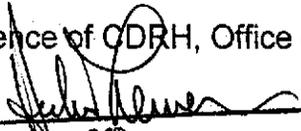
(Division Sign-Off)
Division of Reproductive, Abdominal,
and Radiological Devices

510(k) Number K083160