KO91299 page 1 gl 2

# **STED Summary of Substantial Equivalence**

#### SUBMITTER INFORMATION

- A. Company Name: Spectranetics Corporation, Inc.
- B. Company Address: 9965 Federal Blvd Colorado Springs, CO 80921

JUL - 1 2009

- C. Company Phone: 719-633-8333/ 1-800-633-0960
- D. Company Facsimile: 719-447-2040

#### **DEVICE IDENTIFICATION**

- A. Device Trade Name: Spectranetics Turbo-Tandem<sup>™</sup> System
- B. Device Common Name: Percutaneous Laser Ablation Catheter
- C. Classification: Catheter, Percutaneous
- D. Device Class: Class II (per 21 CFR 870.1250)
- E. Device Code: DQY
- F. Establishment Registration Number: 3007284006

## PERFORMANCE STANDARDS

Performance standards do no currently exist for these devices. None established under Section 514.

#### DEVICE DESCRIPTION

The Turbo-Tandem<sup>™</sup> System (Laser Guide Catheter with Laser Ablation Catheter) is a laser ablation catheter constrained within a guiding catheter to facilitate the offset (biased position) of the laser ablation catheter. The guiding catheter portion of the Turbo-Tandem System is used to offset the distal end of the incorporated laser catheter from the central plane of the vessel lumen allowing for circumferential guidance and positioning of the laser catheter within the vessel. The Turbo-Tandem System is 7F sheath compatible with a maximum crossing profile of 0.160" (4.0mm) with the laser catheter extended or offset position. The incorporated laser catheter is constructed of multiple optical fibers arranged circumferentially around a guidewire lumen and has a fiber optic surface area similar to a 2.0mm laser catheter.

#### INDICATION FOR USE

Indicated for atherectomy of infrainguinal arteries.

K091299 page 2022

### SUBSTANTIALLY EQUIVALENT DEVICES

In Spectranetics' opinion, the Turbo-Tandem System is believed to be substantially equivalent to the following predicate devices currently in interstate commerce with respect to comparable features, materials of construction and intended use.

- Spectranetics Turbo Elite<sup>™</sup> K071227
- Spectranetics Turbo Booster™ K071226

Labeling, packaging, and sterilization of the Spectranetics Turbo-Tandem System has not changed from the predicate devices listed above.

#### SUMMARY OF STUDIES

Spectranetics performed device integrity testing to support that the Turbo-Tandem System is equivalent to the predicate devices. All device integrity test results for the Spectranetics Turbo-Tandem System met specified requirements.

## CONCLUSION (STATEMENT OF EQUIVALENCE)

Through data and information presented, numerous similarities support a determination of substantial equivalence, and therefore market clearance of the Spectranetics Turbo-Tandem<sup>™</sup> System through this STED Premarket Notification.

**DEPARTMENT OF HEALTH & HUMAN SERVICES** 



Public Health Service

JUL - 1 2009

Food and Drug Administration 9200 Corporate Boulevard Rockville MD 20850

Ms. Gracie Murguia Regulatory Affairs and Compliance Supervisor Spectranetics Corp. 9965 Federal Drive Colorado Springs, CO 80921

Re: K091299

Turbo-Tandem System Regulation Number: 21 CFR 870.1250 Regulation Name: Catheter, Percutaneous Regulatory Class: Class II Product Code: DQY Dated: May 1, 2009 Received: May 4, 2009

Dear Ms. Murguia:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Page 2 - Ms. Gracie Murguia

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to <u>http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm</u> for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97): For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address

http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm.

Sincerely yours,

onna R. Lochner

Bram D. Zuckerman, M.D.

Division of Cardiovascular Devices Office of Device Evaluation Center for Devices and Radiological Health

Enclosure



Premarket Notification Number: TBD K091299

Device Name: Turbo-Tandem<sup>™</sup> System

# **Indication for Use**

Spectranetics

Turbo-Tandem<sup>™</sup> System is indicated for atherectomy of infrainguinal arteries.

# (PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Je Mal

(Division Sign-Off) Division of Cardiovascular Devices

510(k) Number <u>K091299</u>

Prescription Use:  $\sqrt{}$  (Part 21 CFR 801 Subpart D)

OR

Over the Counter Use: (21 CFR 807 Subpart C)