

JUL 30 2009

510(k) Summary

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirement of SMDA and 21 CFR 807.92.

1.0 submitter's information

Name: Andon Health Co., Ltd.
 Address: No 31, Changjiang Road, Nankai District, Tianjin,
 P.R. China
 Phone number: 86-22-6052 6161
 Fax number: 86-22-6052 6162
 Contact: Liu Yi
 Date of Application: 06/30/2009

2.0 Device information

Trade name: Fully Automatic Electronic Blood Pressure Monitor
 Common name: Noninvasive blood pressure measurement system
 Classification name: Noninvasive blood pressure measurement system

3.0 Classification

Production code: DXN- Noninvasive blood pressure measurement system.
 Regulation number: 870.1130
 Classification: II
 Panel: Cardiovascular

4.0 Predict device information

Manufacturer: Andon Health Co., Ltd.
 Device: KD-7909 Fully Automatic Electronic Blood Pressure Monitor
 510(k) number: K090768

5.0 Device description

KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 6.1023 inches to 9.8425 inches.

It is designed and manufactured according to ANSI/AAMI SP10--manual, electronic or automated sphygmometers.

The operational principle is based on oscillometric and silicon integrate pressure sensor technology, the result will be shown on a LCD with an electronic interface module, the result can also be classified and displayed by the function of blood pressure classification indicator, the memory capability is 60 times. If any irregular heartbeat is detected, it can be shown on the LCD.

6.0 Intended use

KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 6.1023 inches to 9.8425 inches.

The intended use and the indication for use of KD-738 and KD-739, as described in its labeling are the same as the predict device KD-7909.

7.0 Summary comparing technological characteristics with predicate device

| Technological Characteristics | Comparison result |
|--------------------------------------|--------------------------|
| Design principle | Identical |
| Appearance | Similar |
| Patients contact Materials | Identical |
| Performance | Similar |
| Biocompatibility | Identical |
| Mechanical safety | Identical |
| Energy source | Identical |
| Standards met | Identical |
| Electrical safety | Identical |
| EMC | Identical |
| Function | Identical |

8.0 Performance summary

KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor conforms to the following standards:

- IEC 60601-1, Medical Electrical Equipment - Part 1: General Requirements for Safety, 1988; Amendment 1, 1991-11, Amendment 2, 1995.
- IEC 60601-1-2, Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral standard: Electromagnetic Compatibility - Requirements and Tests (Edition 2:2001 with Amendment 1:2004; Edition 2.1 (Edition 2:2001 consolidated with Amendment 1:2004)).
- AAMI SP10:2002, Manual, electronic or automated sphygmomanometers.
- AAMI / ANSI SP10:2002/A1:2003 --, Amendment 1 to ANSI/AAMI SP10:2002 Manual, electronic, or automated sphygmomanometers.
- AAMI / ANSI SP10:2002/A2:2006 --, Amendment 2 to ANSI/AAMI SP10:2002 Manual, electronic, or automated sphygmomanometers.

9.0 Comparison to the predict device and the conclusion

Our device KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor is substantially equivalent to the Fully Automatic Electronic Blood Pressure Monitor KD-7909 whose 510(k) number is K090768.

The two devices are very similar in the intended use, the design principle, the material, the energy source the functions and the applicable standards. Their appearance are different. The performance parameter of cuff pressure range and overpressure limit are different from the predicted device KD-7909. KD-738 and KD-739 also use a different MCU.

However, the test in this submission provides demonstration that these small differences do not raise any new questions of safety and effectiveness.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

JUL 30 2009

Andon Health Co., Ltd.
c/o Mr. Liu Yi
No 31 Changjiang Road
Nankai District, Tianjin
China 300193

Re: K092045
Trade/Device Name: KD-738, KD-739 Fully Automatic Electronic Blood Pressure
Monitors
Regulation Number: 21 CFR 870.1130
Regulation Name: Noninvasive Blood Pressure Measurement System
Regulatory Class: Class II (two)
Product Code: DXN
Dated: June 30, 2009
Received: July 06, 2009

Dear Mr. Yi:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Page 2 – Mr. Liu Yi

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to <http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm> for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,



Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Statement of Indications for Use

510(k) Number: K092045

Device name: KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor

Indications for use:

KD-738 and KD-739 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 6.1023 inches to 9.8425 inches.

Prescription use _____ AND/OR Over-The-Counter Use YES
Part 21 CFR 801 Subpart D) (21 CFR 807 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-COUNTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)



(Division Sign-Off)

Division of Cardiovascular Devices

510(k) Number K092045

Page 1 of 1